




J-359

Prolapse Repair Kit

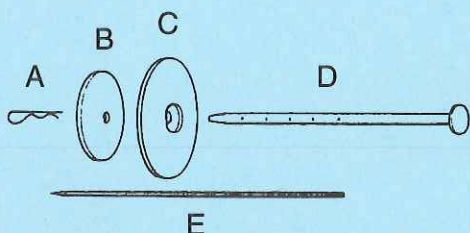


List of Parts

Piece No.	Description
A	Cotter Pin
B	2½" Diameter Plastic Button
C	3" Diameter Plastic Button
D	6" Plastic Trocar
E	7" Stainless Steel Pin


A kit for the correction of preparturient vaginal prolapse.

INSTRUCTIONS FOR USE

1. Administer an epidural.
2. Cleanse and reduce prolapsed vagina back into its normal position.
3. Assemble the JorVet Prolapse Kit as shown (Figures 2 & 3). After soaking in cold sterilization:

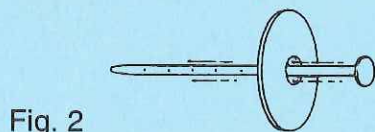

A) Insert 6-inch plastic trocar through 3-inch larger button. The head of the trocar will seat into countersunk hole in button.

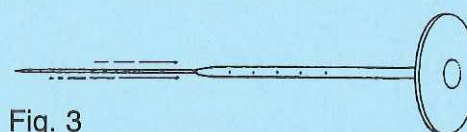

B) The 7-inch stainless rod is inserted in the plastic trocar. The blunt end is inserted first with the sharp-pointed end facing out.

4. The assembled product is placed in the vagina with the sharp point protected from accidental puncture (figure 4). The stainless pin is pointed upward and thrust through the top of the vaginal wall. The direction of the pin should be 4-inches from mid-line. It will penetrate the sacrosiatic ligament and the biceps femoris muscle before reaching the skin. A scalpel blade may be needed to help the pin penetrate the skin. You will exit about 6-inches caudal to the tuber coxae. (Note: Do not go straight upward or risk penetration of the rectum.)

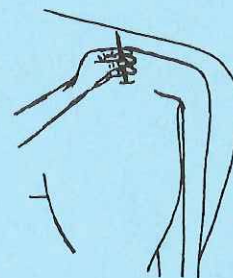


Fig. 4

5. Remove the stainless trocar from the plastic trocar.
6. The smaller 2½" plastic button is placed on the protruding trocar (figure 5). Push down until the vaginal wall is held tightly, allowing one finger width of gap inside the vagina.

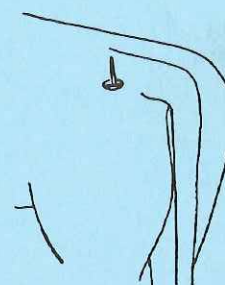


Fig. 5

7. Insert the cotter pin through the hole in the protruding trocar closest to the outside button (figure 6).

In severe cases, it may be necessary to place buttons on both right and left sides.

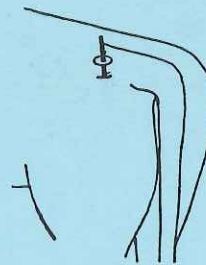


Fig. 6

8. Cut the plastic trocar off 1-inch above the cotter pin (figure 7). This is done to avoid injury to other animals. The outside button should be cleansed and fly repellent used if deemed necessary. Systemic antibiotics may also be deemed necessary.

The button is left in place until calving has taken place and a normal delivery should be expected.

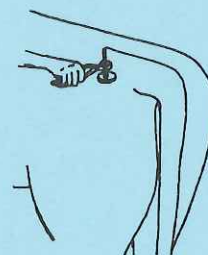


Fig. 7

9. One week after calving, remove the cotter pin and outside button and withdraw the trocar and button from inside the vagina (figure 8).



Fig. 8

NOTE OF CAUTION

This kit is for vaginal prolapse only and other methods may be necessary depending on the animal's size and situation.

Catalog #:

J-359 Complete JorVet Prolapse Kit



Jorgensen Laboratories, Inc.

Loveland, CO 80538