



We make caring for animals easier.

CARE OF NEW INSTRUMENTS

ON RECEIPT:

1. Clean carefully, using instrument cleaner to remove factory residue.
2. Rinse with de-ionised water and dry thoroughly.
3. Re-oil box-locks and cutting edges with instrument oil.
4. For sterilisation, instruments with ratchets should be closed to the first ratchet position only, to avoid tension and consequent damage due to heating and cooling during sterilisation.

AFTER USE:

Clean with instrument cleaner, rinse and dry as soon as possible after surgery. If cleaning will be delayed, immerse instruments in cold water for a maximum of 2 hours. Follow steps 2 to 4 above before each sterilisation.

STORAGE:

Always store instruments in a dry place. Guard against accidental damage to instrument surfaces. Avoid dropping or banging instruments against each other.

REGULAR MAINTENANCE:

To prevent rusting, spotting and staining, instruments should be immersed in Surgical Instrument Lubrication (203 731) on a regular basis. Any stains or corrosion may be removed by soaking in Surgistain (207 482)

Shoof International Ltd, Private Bag 522,
Cambridge 3450, NEW ZEALAND
Call free 0800-800-801 www.shoofvet.co.nz