

Cowslips™

FITTING INSTRUCTIONS

PLEASE NOTE IF YOU DO NOT ADHERE TO THESE INSTRUCTIONS THE MANUFACTURER WILL NOT ACCEPT ANY RESPONSIBILITY FOR POOR ADHESION.

1. Place liquid into your pocket on a cold day to allow heating, or if you have a heat gun please warm your Cowslips.

2. The sole and the wall of the hoof must be thoroughly cleaned with a rasp or rotary sander removing all debris and old horn. The claw must be Very Dry - heat gun or dryer to remove excess moisture..

3. Trial fit the shoe before applying adhesive. Some trimming of the healthy claw may be necessary.

4. Put Liquid into the Cowslips first, followed by the Powder. Thoroughly mix the adhesive within the shoe to achieve an even paste consistency.





5. Fit the shoe on healthy claw. Position firmly by tapping the toe of the shoe. NB. This is important to achieve the ideal positioning. Sole of the shoe should be parallel to sole of claw.

6. Clean off any excess adhesive so that adhesive does not touch hoof/skin junction.



7. Leave foot in raised position for several minutes until adhesive has set.

8. Place foot on ground and keep the animal standing for a few minutes.

9. Animal is now free to walk uninhibited.

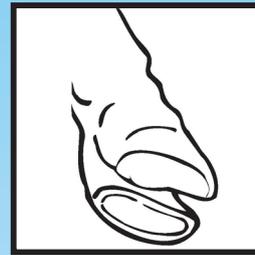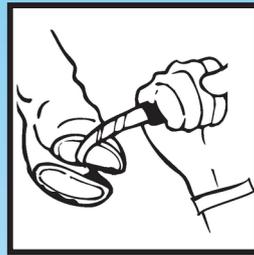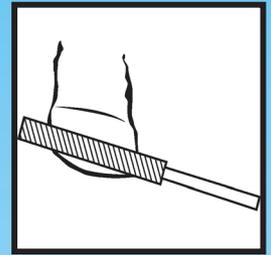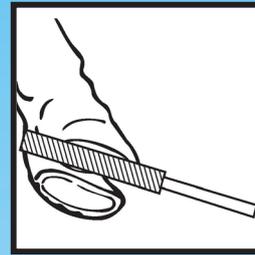

Cleaned and prepared claw ready for trial fitting of the Cowslip.

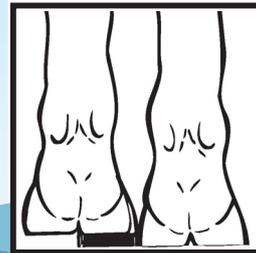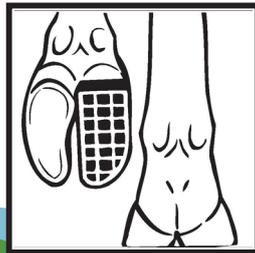



1

ORIGINAL

For use on Jerseys, Guernseys, Heifers and Ayrshires.

- 116mm long
- 60mm wide
- 16mm high



2

PLUS

For use on Holsteins, Friesians and Beef Breeds.

- 129mm long
(13mm longer than the original)
- 61mm wide
(1mm wider than the original)
- 17mm high
(1mm higher than the original)



3

XL

For use on large/abnormal claws, North American Holsteins.

- 142mm long
(26mm longer than the original)
- 62mm wide
(2mm wider than the original)
- 18mm high
(2mm higher than the original)

BENEFITS OF THE COWSLIP

Cowslips™

- The glue is mixed inside the shoe: simple, clean and easy.
- The Cowslips increase the mobility of the affected animal. It allows for increased milk production and weight gain.
- Cowslips can be easily cut and adapted for different shapes of feet and treatments.
- The unique form of the Cowslips assures good surface contact with the claw meaning it stays on for 4-6 weeks before shedding naturally.