



Equine Nebuliser Hippomed Set 224 629

INSTRUCTIONS FOR USE



00000



Lift the Cloud Chamber off the AirOne Flex Nebuliser unit



Measure 60ml of Saline Solution



Pour the Saline solution straight into the ultrasonic chamber (max 60ml)





If medication is being used place the medication into the plastic cup and place on top of the saline solution (do not mix the two)



Put cloud chamber back onto the unit



Attach the desired length hose to the ultrasonic unit



If using the shorter hose, secure mask on both sides with rubber attachments



Supplied by Shoof International Ltd. www.shoofdirect.co.nz