



Quicksplint Leg Splint

A clear plastic tarsal splint for cats and dogs designed for quick, accurate fitting.

Buster Quick Splints are designed to support the distal leg from either the lateral or medial side when properly padded with a soft bandage and cut to fit.

They can be used to remedy a variety of problems, including some types of fractures, luxations, soft tissue trauma, post surgical care, first aid situations etc.

The Quick Splints are liberally perforated to accommodate drainage and ventilation. Grooves in the thermoplastic are designed to facilitate easy cutting.

Type	Left	Right
foreleg small	212 703	212 704
foreleg medium	212 705	212 706
foreleg large	212 707	212 708
hindleg small	212 697	212 698
hindleg medium	212 699	212 700
hindleg large	212 701	212 702

Apply an approx. 3cm strip of adhesive tape to both lateral and medial sides of the tarsus, and extend the strip 15-20cm extra creating a stirrup.

Apply the necessary padding and wrap the leg with roll gauze or stretch gauze.

Select the appropriate splint size and cut it to fit the leg. The splint is easily shortened at each marked groove by cutting with heavy shears. Remove sharp edges. Secure the splint to the leg by reflecting the tape stirrup onto the splint and taping it in place.

Tape the entire splint to the leg using adhesive tape. Leave areas free of tape for airing.