


Cowslips



Features

- The glue is mixed inside the shoe for simple, clean and easy application.
- Increases the mobility of the affected animal and allows for increase in milk production and weight gain.
- Can be easily cut and adapted for different shapes of feet and treatments.
- The unique form assures good surface contact with the claw allowing it to stay on for 6-8 weeks before shedding naturally.

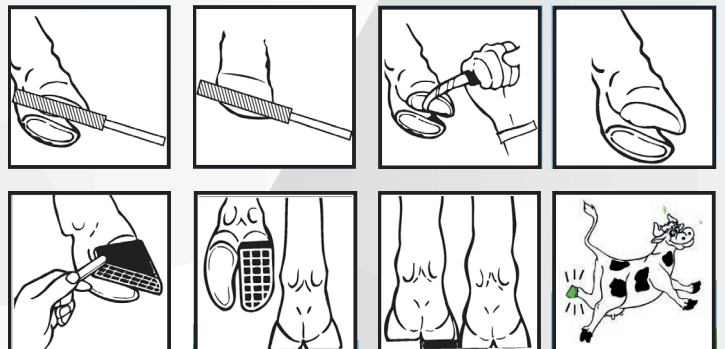

Cowslips

Product Information:

There is no doubt that Cowslips are one of the leading prosthetic device for lameness in New Zealand cows. Cowslip was developed in UK by Dr Ernie Logan in the late 1980s, and is now used worldwide.

Before fitting a Cowslip the hoof should be lifted and cleaned, and the cause of the lameness determined. (A Cowslip should not be applied until the damaged hoof area is identified, as accidental application to the damaged claw could exacerbate the injury.) Application of the Cowslip to the healthy claw allows relief of load-bearing on the damaged claw so that healing can occur. The Cowslip will slough-off of its own accord in two or three weeks, by which time the affected claw should have recovered. Use the Cowslip 'Original' for all regular-size cows, 'Plus' for large and heavy cows, and 'X-large' for very large and heavy animals.

| Code | Product |
|--------|---|
| 211580 | Original Mixed L+R 4-pack End of line |
| 211582 | Original Right only 4-pack End of line |
| 211583 | Cowslips Original Mixed L+R 10-pack |
| 211584 | Original Left only 10-pack |
| 211585 | Original Right only 10-pack |
| 211589 | Cowslips Plus Mixed L+R 10-pack |
| 211590 | Cowslips Plus Left only 10-pack |
| 211591 | Cowslips Plus Right only 10-pack |
| 211592 | Cowslips X-Large Mixed L+R 4-pack |

| Code | Product |
|--------|-----------------------------------|
| 211600 | Cowslips Powder Single App'n each |
| 211601 | Cowslips Liquid Single App'n each |



1

ORIGINAL

For use on Jerseys, Guernseys, Heifers and Ayrshires.

- 116mm long
- 60mm wide
- 16mm high



2

PLUS

For use on Holsteins, Friesians and Beef Breeds.

- 129mm long
(13mm longer than the original)
- 61mm wide
(1mm wider than the original)
- 17mm high
(1mm higher than the original)



3

XL

For use on large/abnormal claws, North American Holsteins.

- 142mm long
(26mm longer than the original)
- 62mm wide
(2mm wider than the original)
- 18mm high
(2mm higher than the original)



Cowslips[®]